

DIAMONDS ARE A DENTIST'S BEST FRIEND

Adjust. Finish. Polish.

BruxZir™

ADJUSTMENT & POLISHING

New &
Improved



Benefits

- Adjusts restorations without creating microfractures
- Gently removes surface irregularities and helps prevent staining
- Two NEW polishers quickly create a natural, high shine

*Embedded with a rich concentration of fine diamonds
designed for safe and effective use on zirconia*

BRUXZIR ADJUSTING BURS

Always use light pressure with water and air spray when adjusting BruxZir restorations to avoid microfractures.



A **round bur** is used for adjusting a fossa.



A **tapered bur** is most effective for adjusting cusps or proximal contacts.



A **football-shaped bur** is most effective for adjusting the occlusal surfaces of posterior teeth and the lingual surfaces of anterior teeth.

BRUXZIR POLISHING CUPS



Using light pressure and no water, begin pre-polishing with the **green cup** to remove abrasions left by the diamonds during adjustments. The green cup can also be used alone for minor adjustments if you don't want to use a diamond.



Continue pre-polishing with the **brown cup** until a more glossy look appears on the adjustment areas.



Use the **white cup** with light to medium pressure to achieve a "wet" high shine.

BRUXZIR POLISHING DISCS



The **brown disc** gently removes surface defects. Use in circular motions at 7,000–10,000 RPM to make minor adjustments to occlusal surfaces without affecting the anatomy.



The **blue polishing disc** is ideal for difficult-to-reach areas. Use light to medium pressure at 7,000–10,000 RPM to achieve a natural, high-shine finish in as little as 20 seconds.

Speak with a Glidewell representative to order your kit today!



800-212-9080

mail@glidewell dental.mx

mx.glidewell dental.com